
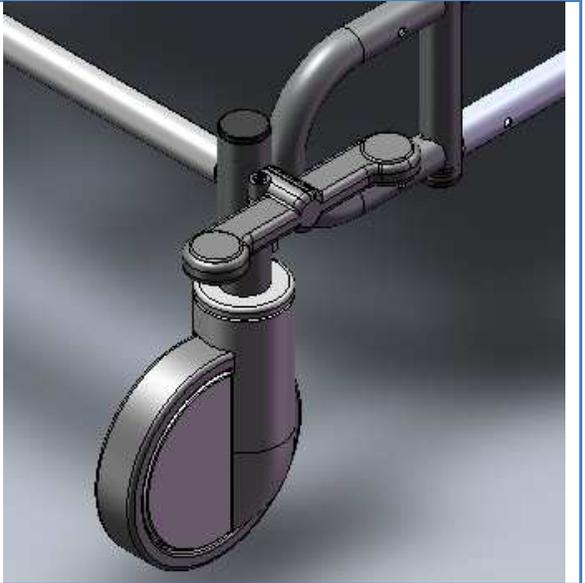
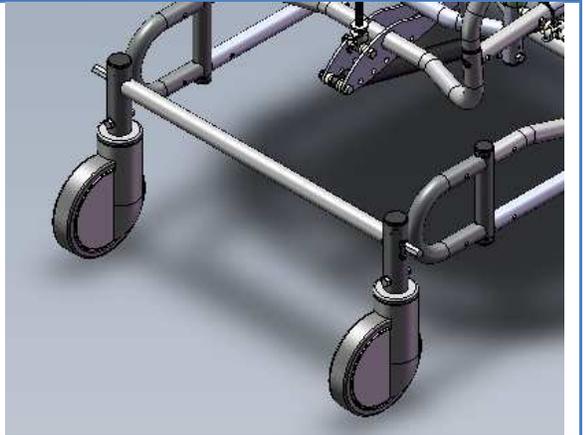
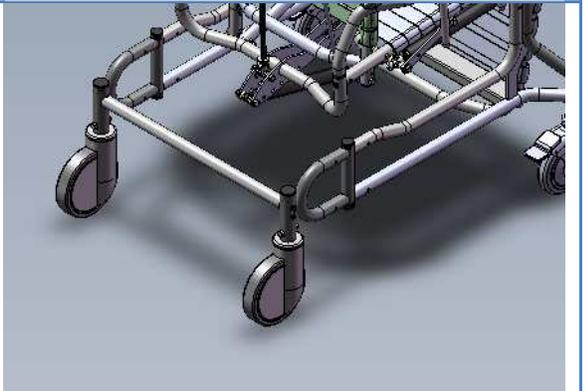


Parts Required: Control Rod (width specific)

Tools Used: 5/32" Allen Key
Rubber Mallet

Step	Execution	Figures
1	Using a 5/32" Allen Key, loosen the set screws and remove the left and right MDLC PEDAL. (Set aside set screws for re-use)	
2	Slide out the original control rod. You may need a rubber mallet to tab out the hex rod if resistant.	
3	Insert replacement control rod the same way it was removed in the above step.	

- 4 Re-attached left and right MDLC PEDALS the same way they were removed in Step # 1 and tighten with 5/32" Allen Key.

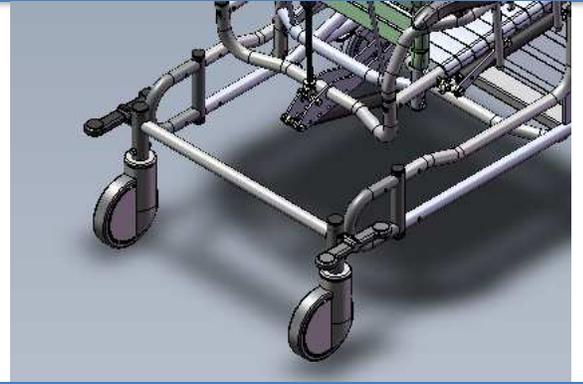

For more information, please contact Broda Customer Care at 1-800-668-0637