


785 Legrest Replacement

Parts Required:

- Legrest Frame (width & strap color specific)


Tools Used:

- ½" Wrench

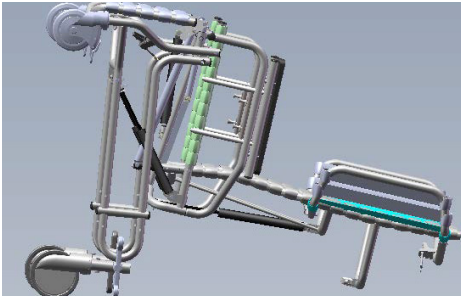


Step 1:

Using a #2 Robertson screwdriver, remove the screw from the Tilt Handle.

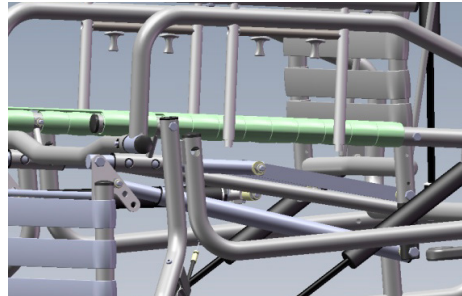

Step 2:

**Before removing any hardware, keep in mind the sequence of hardware, for the installation of the replacement legrest.* Standing at the front of the chair, the elevation rod is on the right hand side, positioned under the attachment rod. Using two ½" wrenches remove the ⅝-18 hex nut from the outer side of the elevation rod. Remove elevation rod from legrest.

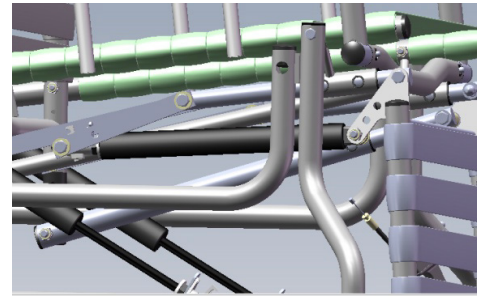

Step 3:

Rotate the cable clip free from both cables. Remove the end of the cable from the actuator lever.



Step 4:

Using a #2 Robertson screw driver, attach the new Double Cable Handle to the recline bar.



Step 5:

Slide the attachment rods through the legrest guide tube bushings found underneath both sides of the seat. Install the two attachment rods in the top hole of the flange tab using two ½" wrenches, the same way it was removed in step # 5.



Step 6:


Install the replacement legrest by sliding one washer on the bolt, assemble the bolt through the elevation rod, slide on a second washer, spacer, then third washer and slide through the second hole of the flange tab. Use two ½" wrenches and using the same sequence of hardware, as removed in step # 3.



brodaseating.com

info@brodaseating.com

[1-844-552-7632](tel:1-844-552-7632)

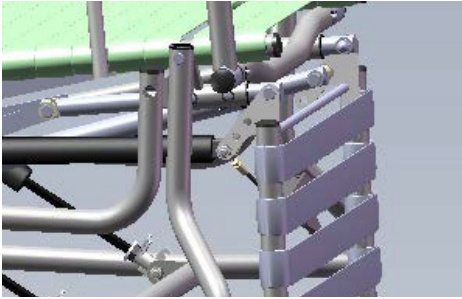

Step 7:

Using two ½" wrenches, install the PAB5 cylinder to the bottom hole in the flange tab, the same way it was removed in step # 4.

Note: The elevation rod is date and size specific, be sure to supply a serial upon ordering a replacement.

For more information:

Please contact Broda Customer Care at 1-800-668-0637