

BRODA®

brodaseating.com

info@brodaseating.com

[1-844-552-7632](tel:1-844-552-7632)

BACK PAD INSTALLATION



Step 1:

Position the back pad with the mesh material against the vinyl straps. The tags should be positioned at the top of the back frame and the narrow end of the pad towards the bottom of the back frame.



Step 2:

The back pad does not have any straps to secure, therefore it will sit freely in an upright position against the back frame.

****Note the back pad and covers are width and height specific. The covers are color specific.***

For more information:

Please contact Broda Customer Care at 1-800-668-0637