


Parts Required:

- Comfort Company Lateral Torso Support

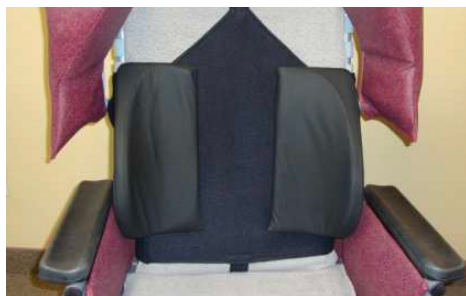


Step 1:

Lateral Supports attach to the Velcro pad and can be repositioned anywhere on the pad to provide individualized support for residents. Position supports in most the suitable and comfortable position for our resident.



Step 2:

Fasten the pad around the top and bottom of the back of the chair by joining the straps together using the extension piece and inserting the extension piece through the loop. Do not fasten around cables, cylinders or frame supports.

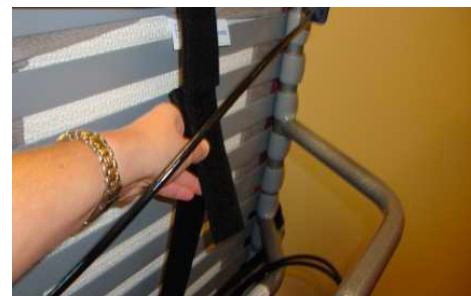

Step 3:

Tighten as needed.



Step 4:

Fasten the pad around the sides of the chair. There are three straps to attach, on the side. Insert the straps through the loop to attach. Do not fasten around cylinder, cables or frame supports. Tighten as needed.

For more information:

Please contact Broda Customer Care at 1-800-668-0637