
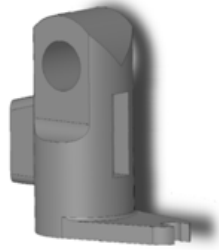


Parts Required:

- Actuator
- Lever
- Cable Clip

Tools Used:

- 17mm Wrench

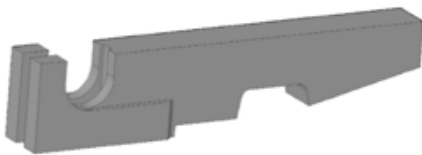

Step 1:

Hold the lever in your right hand, with the long flat side, and the “u” cut out pointing upward.

Step 2:

Hold the head of the actuator in your left hand with the tail portion pointing to the right and the rounded head pointing up.

Step 3:

Slide the small end of the pin through the hole, bringing it through just until it becomes flush with the side of the actuator.



Step 4:

Holding the pin in place screw the cylinder, with cable clip assembly, into the actuator. Make sure the pin stays in place at all times. Tighten the actuator on the cylinder until the pin reaches the top of the hole in the actuator. Do not over tighten.

Note: If Cable Clip installation is required please contact customer service for instruction.

Step 5:

Use a 17 mm wrench to tighten the nut that will be located just underneath the attached actuator.

Note: The Cable Clip will swing freely.

For more information:

Please contact Broda Customer Care at 1-800-668-0637