


### Parts Required:

- Calf Pad (width & color specific)

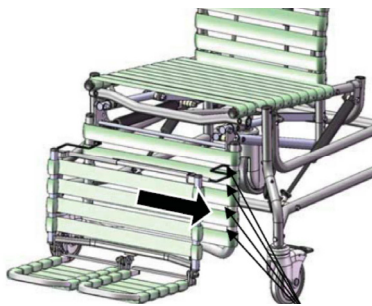

#### Step 1:

Before starting remove footrest completely from leg rest on chair.



#### Step 2:

Take the Calf Pad and place it over top of the calf portion of the footrest having the black fabric with Velcro facing the back and upward.




#### Step 3:

Take the black piece of fabric and wrap it over top of the top strap. Tuck the fabric underneath the second strap on the back and secure it to the Velcro located on the back of the pad.



#### Step 4:

Take the two sets of straps with buckles on the ends and wrap them around the back of the footrest and secure. Tighten as desired.

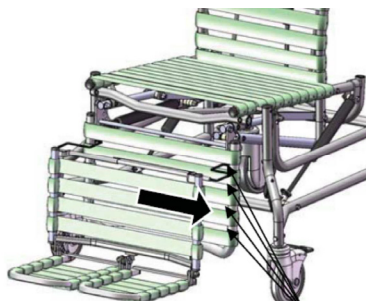

#### Step 5:

Place the footrest back on chair at desired length.

### For more information:

Please contact Broda Customer Care at 1-800-668-0637