


Parts Required:

- Number 3 Spacer (thinner)
- Number 4 Spacer (thicker: 5/16-18)
- 1 3/4 Long Bolt
- 5/16 Nylock Nut

Tools Used:

- 1/2" Wrench

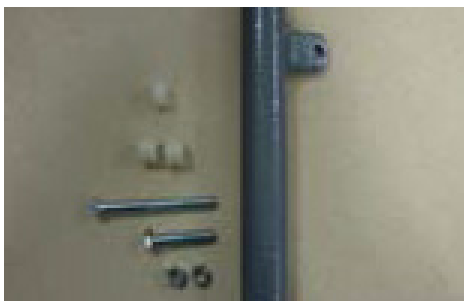

Step 1:

Remove the hardware that attaches the cylinders to the chair frame. Set aside original hardware for further use.



Step 2:

Place the Cylinder Adapter over the wider end of the black cylinder (line up the holes for securing the two pieces together). Secure in place with provided hardware.



Step 3:

Flip both cylinders and re-attach actuator end to the chair using the original hardware. The actuators should be pointing towards the rear of the chair.



Step 4:

Align the tabs on the Cylinder Adapter with the tabs on the base of the chair and fasten with provided hardware.

For more information:

Please contact Broda Customer Care at 1-800-668-0637