

Parts Required:

- Cable x 2 (32", 48", 56" cable lengths)
- Handle

Tools Used:

- #2 Robertson Screwdriver
- 17mm Wrench



Step 1:

Remove the cable tie holding the two cables to the tilt bar.



Step 2:

Using a #2 Robertson screwdriver, remove the screw from the Tilt Handle.



Step 3:

Using a 17mm wrench, loosen the black cylinder nut, on the bottom of both tilt cylinders.

Note: *The current style cable clip includes a washer, which does not require the nut to be loosened, please ignore this step and move onto step 4.*




Step 4:

Rotate the cable clip free from both cables. Remove the end of the cable from the actuator lever.



Step 5:

Using a #2 Robertson screw driver, attach the new Double Cable Handle to the recline bar.

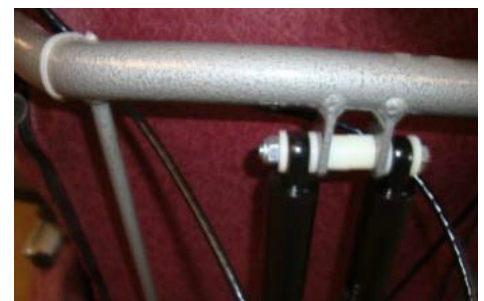

Step 6:

Note: *using cable ties, tie the cables on either side of the cylinders, securing to the back frame. See picture.*



Step 7:

Attach the end of each cable to the actuator levers, the same way removed in step # 5.



Step 8:

Secure Cable Clips to the end of the Cables, attached to the actuator.



Step 9:

Using a 17mm wrench, tighten the black cylinder nut, on the bottom of each cylinder.

Note: *The current style cable clip includes a washer, which does not require the nut to be loosened, please ignore this step.*

For more information:

Please contact Broda Customer Care at 1-800-668-0637