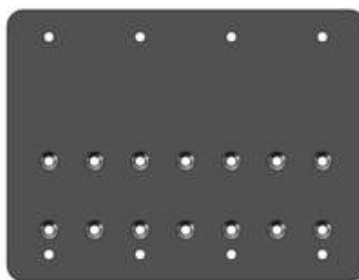

Parts Required:

- FOOTBOARD-587V3 (Black or Charcoal Vinyl; width specific)
-



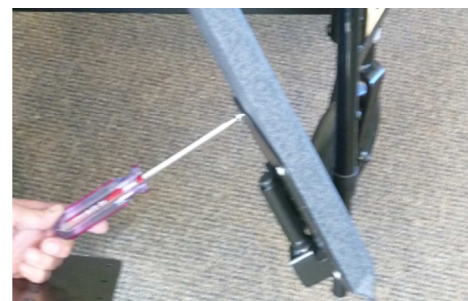
Step 1:

Footboard is designed to be bolted onto one of the footplates. The footboard will have four "X"s cut into the vinyl where the mounting holes are located.



Step 2:

Line the holes on the footboard up with holes on the footrest.



Step 3:

Secure with four 10-32 x 3/8" pan quan zinc screws (pan head screws).



Step 4:

The footboard should be secure to the footplate and able to flip up and down.



Picture continued from step # 4.

For more information:

Please contact Broda Customer Care at 1-800-668-0637