


Parts Required:

- F-ROD Foot Rod (width specific)
- F-SPRING Footrest Spring 3.25" Long


Tools Used:

- #2 Robinson
- Phillips Screwdriver

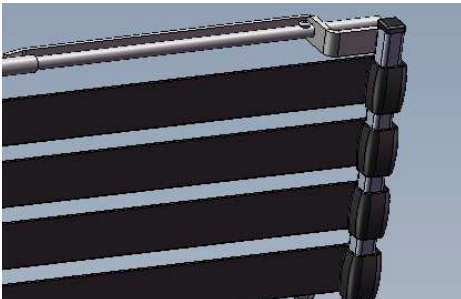


Step 1:

Loose the #8 x $\frac{3}{8}$ Pan quad self-tapping screw attached to the footrest rod using a #2 Robinson or Phillips screw driver to completely remove the defective footrest rod. (Set screw aside for re-use).

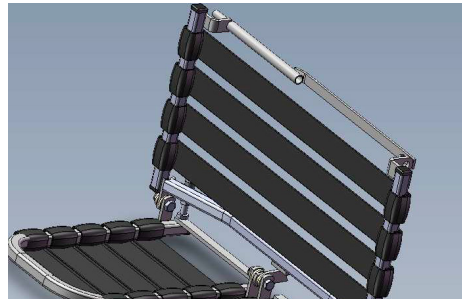


Step 2:

Slightly turn the footrest on an angle to slide the footrest spring out of the frame portion, where the footrest rod was inserted and discard.

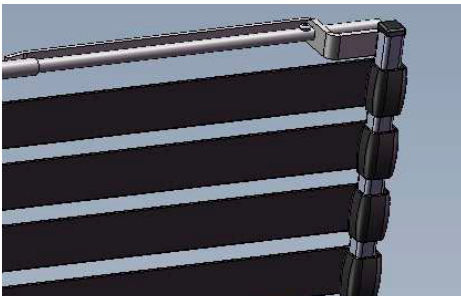

Step 3:

Slide the replacement footrest rod into the cylinder tube, with the hole facing upwards, the same way it was removed in Step #1.

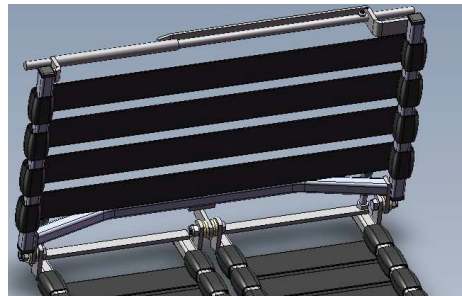

Step 4:

Insert and tighten the #8 x $\frac{3}{8}$ Pan quad self tapping screw the same way it was removed in Step #1.

For more information:

Please contact Broda Customer Care at 1-800-668-0637