

Parts Required:

- IV Pole & Holder

Tools Used:

- $\frac{7}{16}$ " Wrench



Step 1:

Insert the two $\frac{1}{4}$ "-20 x 1 $\frac{3}{4}$ " long hex cap screws, through the pass holes located on the mounting bracket.




Step 2:

Place mounting bracket on one of the rear caster posts, just above the welded cross bars.



Step 2 (continued)

Place the clamp on the opposite side of the caster post & thread the hex cap screws, installed in the step one, through the clamp. Use $\frac{7}{16}$ " wrench to tighten.

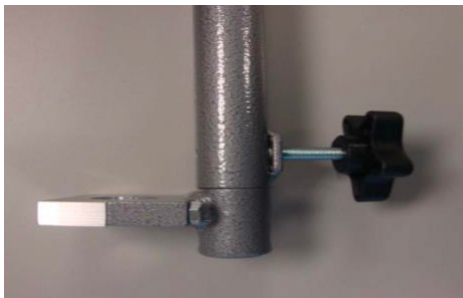

Step 3:

Once the mounting bracket is tightened, slide the larger opening of the IV Pole Holder, over the $\frac{7}{8}$ " stud on the mounting bracket. Use the $\frac{5}{16}$ "-18" Hand Wheel to secure.



Step 4:

At the top, open end, of the IV Pole Holder, insert the IV Pole. Insert the $\frac{5}{16}$ "-18 x $\frac{3}{8}$ " long set screw into one of the weld nuts, at the top of the IV Pole holder. Insert a $\frac{5}{16}$ "-18 Hand Wheel in the Adjacent Weld Nut.



For more information:

Please contact Broda Customer Care at 1-800-668-0637