

Parts Required:

- IV Pole
- IV Bracket
- 1" Block Clamp

Tools Used:

- 7/16" Wrench




Step 1:

To mount the IV pole to the 587 V3, you will require an IV Bracket & Block Clamp.

Step 2:

Place IV bracket on one of the rear caster posts, just above the welded cross bars. Place the clamp on the opposite side of the caster post & thread the hex cap screws, through the clamp. Use 7/16" wrench to tighten.

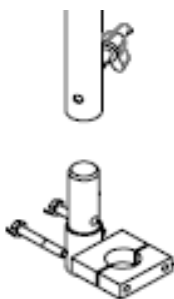

Step 3:

Insert the bottom of the IV Pole onto the mount. Tighten the Handwheel until secure.



Step 4:

To adjust the height of the IV Pole, loosen the handwheel and slide the pole up or down. Tighten the wheel when desired position is achieved.

For more information:

Please contact Broda Customer Care at 1-800-668-0637