


Parts Required:

- IV Mount Adapter
- IV Pole & Holder

Tools Used:

- $\frac{7}{16}$ " Wrench

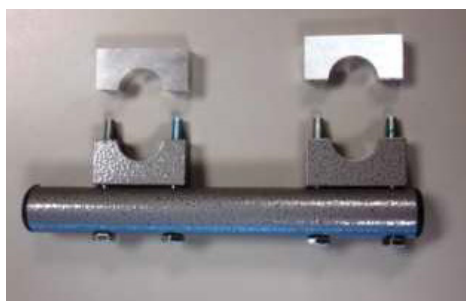

Step 1:

Thread bolts through the IV Pole Adapter and the O2 Holder Block Clamps (powder coated clamps) as depicted.



Step 2:

Place the IV Pole Adapter on the side of the elite base, in front of the mag wheels. The Clamps should fit against the upper & Lower crossbars. Assemble the 1" Aluminum block clamp to the opposite side of the crossbars & thread the bolts from the IV Pole Adapter through. Tighten with a $\frac{7}{16}$ " wrench.



Step 3:

Insert the two $\frac{1}{4}$ "-20 x $1\frac{3}{4}$ " long hex cap screws, through the pass holes located on the mounting bracket.




Step 4:

Assemble the mounting bracket to the IV Pole Adapter. Place the 1" Aluminum Block Clamp on the opposite side of the adapter & Thread the hex cap screws through. Tighten with a $\frac{7}{16}$ " wrench.



Step 5:

Slide the IV Pole Holder, over the Mounting Bracket. Secure using the hand wheel.

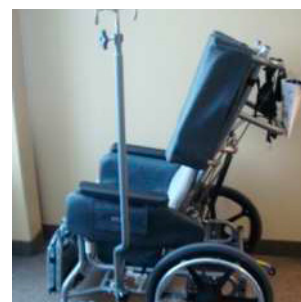

Step 6:

At the top, open end, of the IV Pole Holder, insert the IV Pole. Insert the $\frac{5}{16}$ "-18 x $\frac{3}{8}$ " long set screw into one of the weld nuts, at the top of the IV Pole holder. Insert a $\frac{5}{16}$ "-18 Hand Wheel in the Adjacent Weld Nut.

For more information:

Please contact Broda Customer Care at 1-800-668-0637