


Parts Required:

- Actuator
- Lever
- Cable Clip



Step 1:

The lateral stabilizer kit is width specific. You should have all parts included in the picture.

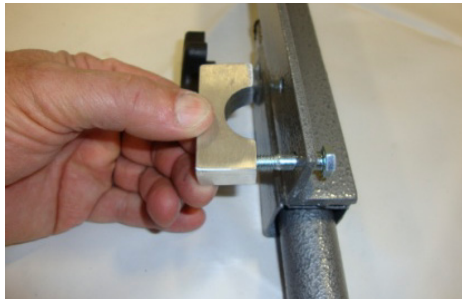

Step 2:

Remove the aluminum clamps from the stabilizer bar.



Step 3:

Position the stabilizer mount across the base frame, under the side rail. The location to mount the stabilizer may vary depending on the model and configuration. Ensure the stabilizer does not interfere with the function of the chair (Eg. Tilt, Recline or Wheels). **Note: Picture shows the stabilizers mounted towards the front of the chair.**



Step 4:

Secure the stabilizer mount to the base frame using the aluminum clamps. Position the clamp on top of the base frame. Insert 1/4"-20 hex bolts upwards, through the holes in the stabilizer and into the clamps. Tighten evenly. Repeat on both sides securing the stabilizer to the base frame.

Note: The position of the bolt.



Step 5:

The stabilizers can be adjusted to flip upward or downward and retract when necessary. To do so, turn the 5 point handle to loosen the stabilizers. Flip the stabilizer up or down and slide back in. Once you've reached the desired position, tighten the 5 point handle. **Note: Stabilizer is shown flipped upwards.**

For more information:

Please contact Broda Customer Care at 1-800-668-0637