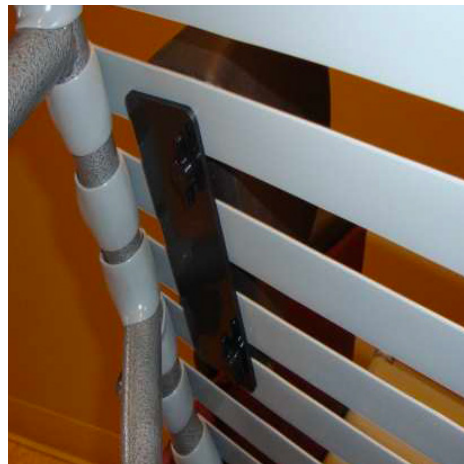

Parts Required:

- Molded Laterals
-



Step 1:

Remove the screws to release the plastic backing from the lateral support.



Step 2:

Position the molded lateral on the vinyl strap in the preferred area and place the plastic behind the vinyl straps so that the holes in the lateral are in line with the holes in the backing. Fasten the plastic backing to the lateral with the screws removed in step 1.



Step 3:

Before the bolster is tightly secured to the chair, it can be positioned horizontally along the straps to achieve optimum positioning for the resident. Once in the desired position, secure the lateral.

For more information:

Please contact Broda Customer Care at 1-800-668-0637