


Parts Required:

- O2 Adapter
- O2 Holder

Tools Used:

- $\frac{7}{16}$ " Wrench

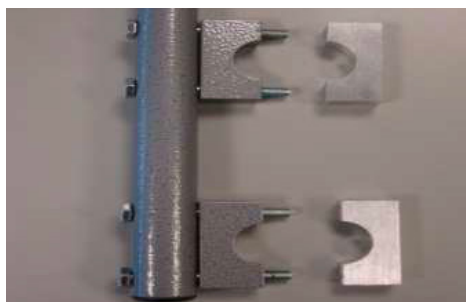


Step 1:

Thread bolts through the O2 Holder Adapter & the O2 Holder Block Clamps (powder coated clamps) as depicted.



Step 2:

Place the O2 Adapter on the side of the elite base, in front of the mag wheels. The clamps should fit against the upper & lower crossbars. Assemble the 1" aluminum block clamp to the opposite side of the crossbars & thread the bolts from the O2 Adapter through. Tighten with a $\frac{7}{16}$ " wrench.

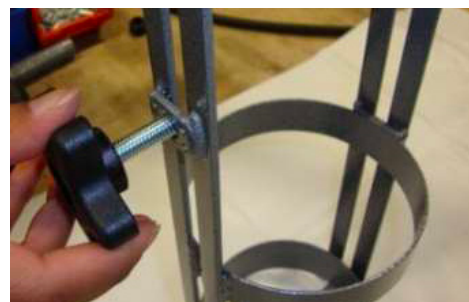


Step 3:

Install a $\frac{5}{16}$ " Hand Wheel into the Threaded Hole.



Step 4:

Insert a $\frac{1}{4}$ "-20" x $1\frac{3}{4}$ " Long Hex Cap Screw, through each of the Non-Threaded holes in one of the painted block clamps.

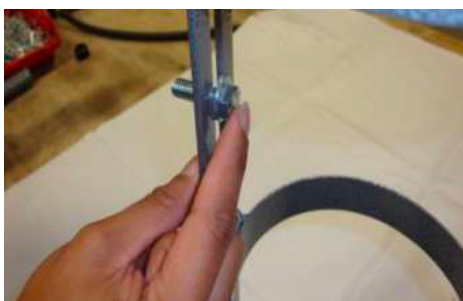

Step 5:

Install a $\frac{5}{16}$ " Flat washer onto a $\frac{5}{16}$ "-18 x 1" long hex cap screw. Pass the hex screw through the upper portion of the $\frac{3}{8}$ " wide slot. (opposite the hand wheel).




Step 6:

Install another $\frac{5}{16}$ " flat washer, follow with a $\frac{1}{4}$ " thick nylon spacer & another $\frac{5}{16}$ " flat washer.




Step 7:

Thread one of the painted block clamps onto the $\frac{5}{16}$ "-18 x 1" Long Hex Screw Assembled Above.

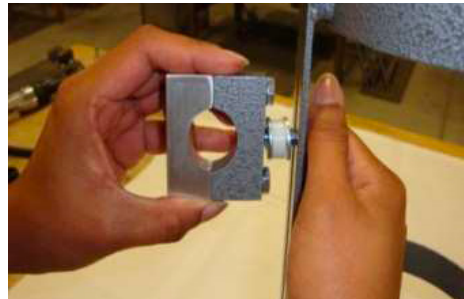

Step 8:

Picture Reflects Above.

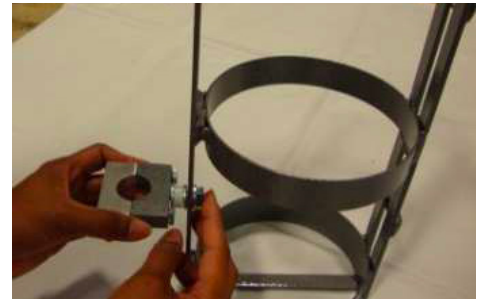

Step 9:

Follow steps 5 through 8 For Lower block clamp.



Step 10:

Using the Aluminum block clamps, attach the O2 Holder to the O2 Holder Adapter. Tighten with a $\frac{1}{2}$ " Wrench.

For more information:

Please contact Broda Customer Care at 1-800-668-0637