


Parts Required:

- Control Rod (width specific)


Tools Used:

- $\frac{5}{32}$ " Allen Key
- Rubber Mallet

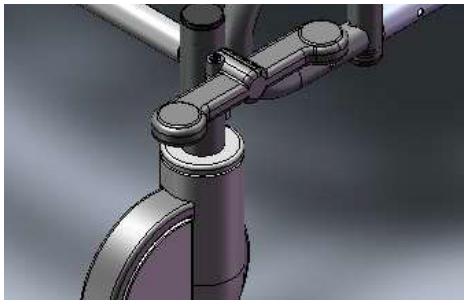


Step 1:

Using a $\frac{5}{32}$ " Allen Key, loosen the set screws and remove the left and right MDLC PEDAL. (Set aside set screws for reuse).

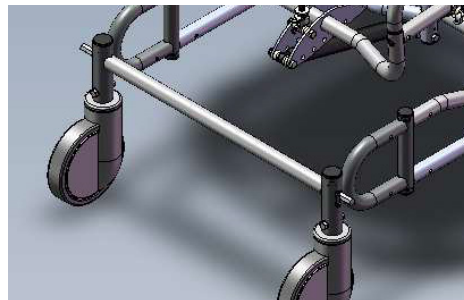


Step 2:

Slide out the original control rod. You may need a rubber mallet to tap out the hex rod if resistant.

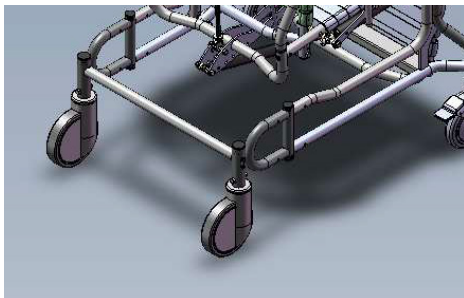

Step 3:

Insert replacement control rod the same way it was removed in the above step.

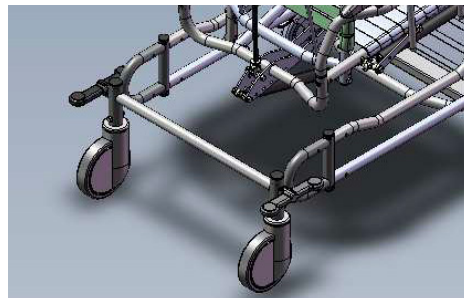

Step 4:

Re-attached left and right MDLC PEDALS the same way they were removed in Step # 1 and tighten with $\frac{5}{32}$ " Allen Key.

For more information:

Please contact Broda Customer Care at 1-800-668-0637