

Parts Required:

- Padded Seat Belt



Step 1:

Place the seatbelt on the resident in their seated position in the chair, with the padded portion centered over the resident's midsection.

Step 2:

Connect the strap buckles behind the chair. Do not fasten around the cable or cylinder behind the chair. Tighten the straps as necessary.

*Add Photo



Step 3:

When using a seat belt, always ensure the resident is in a supervised area and is correctly positioned as to avoid sliding under the seatbelt.

Step 4:

Auto Style buckle also available. Repeat steps 1-3.

For more information:

Please contact Broda Customer Care at 1-800-668-0637