

Parts Required:

- Seat Belt (Option of side or center buckle; Auto buckle is also an option).

**Standard Buckle*




**Auto Buckle*



Step 1:

Unclip the belt to identify the two pieces of the clip mechanism.

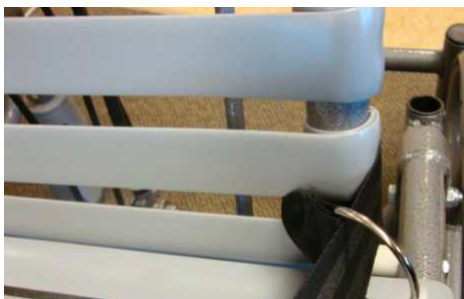

Step 2:

Attaching first the male portion of the belt, insert the belt between the bottom two straps of the seat.



Step 3:

Maneuver the male portion of the clip mechanism through the D ring, located at the end of the belt.



Helpful hint: the three prongs will not go through at once. Only put through two and then you can maneuver the other one in.



Step 4:

Pull the belt tight.



Step 5:

Attaching now the female portion of the belt, insert the belt between the bottom two straps of the seat. With the Velcro facing outward, slide the end of the belt through the two D rings located in the middle of the belt.



Step 6:

Insert the end of the belt in between the two D rings and pull tight. Fasten the Velcro pieces to secure.



Step 7:

**Standard Buckle*

Clip the male and female portions of the clip mechanism together.



**Auto Buckle*

For more information:

Please contact Broda Customer Care at 1-800-668-0637