

BRODA®

brodaseating.com

info@brodaseating.com

1-844-552-7632

Seat Pad Installation



Step 1:

Position your seat pad and cover ontop of the seat frame with the "V" Notch of the pad towards the front of the seat.



Step 2:

Tuck all four straps down the sides of the chair, between the seat frame and arm frame.



Step 3:

Secure both sets of straps and buckles together underneath seat frame. Tighten as desired.



Note: for 785 models, the straps should be secured underneath all moving parts to avoid interference.

For more information:

Please contact Broda Customer Care at 1-800-668-0637