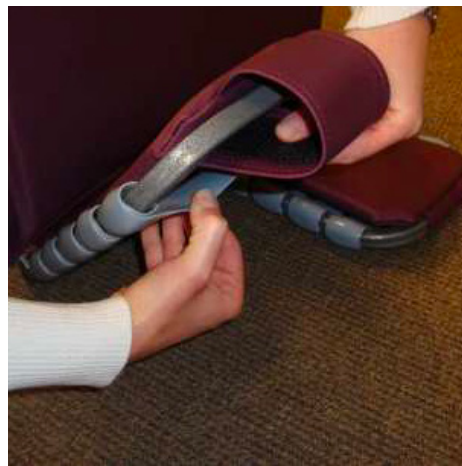

Parts Required:

- SP-SINGLE (width & color specific)
 - SP-SPLIT (width & color specific)
-



Step 1:

Take the sole pad and place over top of the sole portion of the footrest. The long velcro strip needs to be facing the front and all Velcro facing down.



Step 2:

Feed the Velcro strip around the front of the footrest frame and under the strapping. Secure the Velcro on the inside of the sole pad.



Step 3:

Wrap the short ended Velcro piece around the back of the footrest frame and under the strapping, securing it to the Velcro location on the inside of the sole.

For more information:

Please contact Broda Customer Care at 1-800-668-0637