
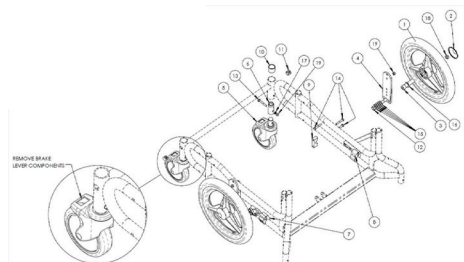


Parts Required:

- Elevating length adjustable ABS footrest

Tools Used:

- 5mm Allen Key
- $\frac{7}{16}$ " Wrench
- $\frac{5}{8}$ " Wrench
- Large Mallet
- Flathead Screwdriver

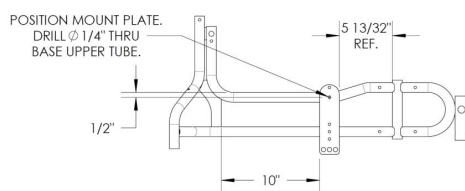

Step 1:

Remove the rear wheels (casters or mags) and all associated parts & fasteners.

Step 2:

Set chair on its side. Position the wheel mounting plate as shown and clamp into position. Using the wheel mounting plate as a template, drill a $\frac{1}{4}$ " hole thru the base upper side rail tube. Repeat on opposite side of base.

Step 3:

Assemble all the parts to the base as shown in the drawing above.



Step 4:

Test that the mag wheels and components have been installed correctly before use. Ensure the mag wheels roll without obstruction when unlocked. Make sure the wheel locks engage the tire enough to bring the mag wheels to a halt when in the locked mode.

For more information:

Please contact Broda Customer Care at 1-800-668-0637