

Tools Used:

- 7/16" Wrench



Step 1:

Lay chair on one side.



Step 2:

Loosen bolts on all four casters and completely remove from the caster posts. If this is a Positioning dual hub model, also remove the casters from the side rail brace and back rail brace. (Keep all hardware to re-use on replacement).



Step 3:

Remove side rail brace and back rail brace if applicable.



Step 4:

Loosen both bolts on each corner of the base, where side rails attach to caster posts to completely remove all casters posts.



Step 5:

Install new caster posts to desired seat height. Insert all fasteners removed in steps above, before tightening. If you tighten before all fasteners are inserted, base may be difficult to align.



Step 6:

If switching caster posts from a Mobility chair (short caster posts) to a Positioning chair (tall caster posts), you will want to install the side rail brace and back rail brace (to lowest hole on caster post) for a dual hub model. If you are switching from a Positioning to a Mobility chair, the side rail brace and back rail brace are not required. Re-install casters the same way they were removed in Step # 2.

For more information:

Please contact Broda Customer Care at 1-800-668-0637